Loading instruction - Viscoject Bio

FEMTIS® Family with plate-haptic




FB-313



FB-313 MF15 FB-313 MF30



FB-313 MF15 T0-T3 FB-313 MF30 T0-T3

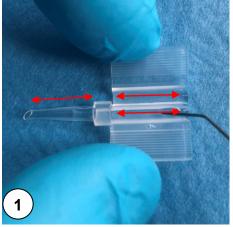


Lubricate the injector tip + loading chamber with viscoelastic solution.




Open the IOL blister away from your body.




Remove IOL holder, align handle towards your body. Always keep it horizontal!

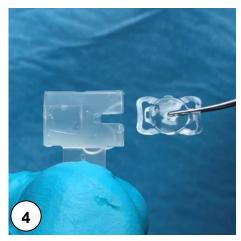

Notch should face to the right side! Remove the IOL from the holder with forceps without excessive force. Use a non-toothed forceps!

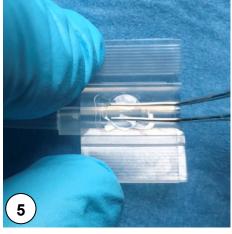

Place the IOL within the cartridge. Pay attention to marking A,B and C. Push the IOL down with forceps while closing the cartridge.

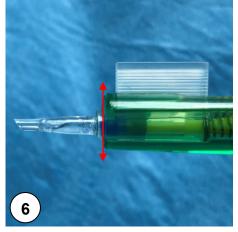

Place the cartridge into the injector. Push the plunger until the blue cushion reaches the tip, followed by short pressure relief. Implant the IOL.

