



Loading instruction - Viscoject Bio


EDOF + MIOL LENTIS® with plate-haptic

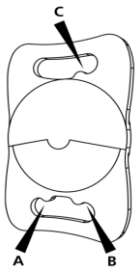

LS-313 MF15
LS-313 MF20

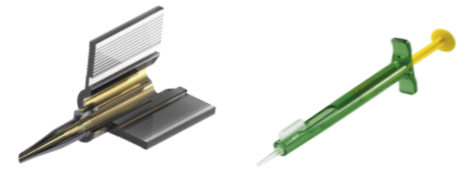


LS-313 MF30

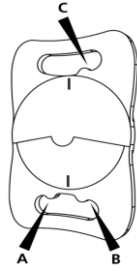

LS-313 MF15 T0-6



LU-313 MF15T/20T/30T
LU-313 MF15TY/20TY/30TY

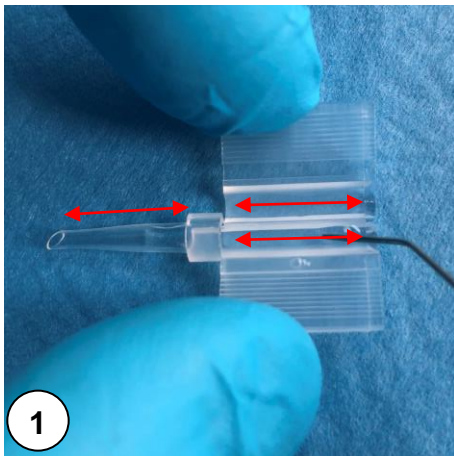

1

Lubricate the injector tip + loading chamber with viscoelastic solution.




2

Open the IOL blister away from your body.




3

Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

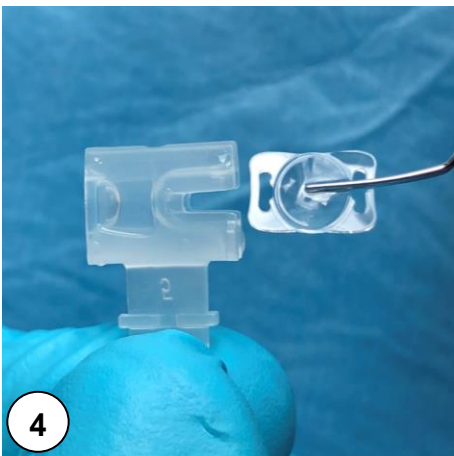


4

Notch should face to the right side! Remove the IOL from the holder with forceps without excessive force. **Use a non-toothed forceps!**

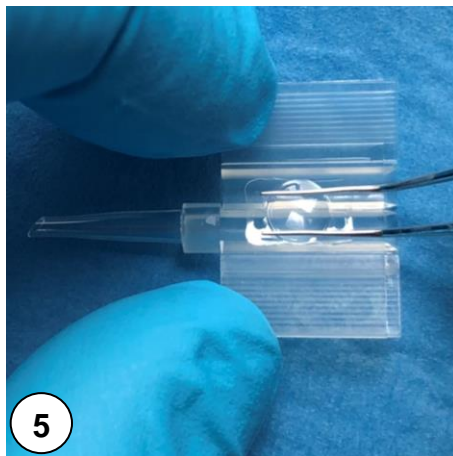

5

Place the IOL within the cartridge. Pay attention to marking A, B and C. Push the IOL down with forceps while closing the cartridge.

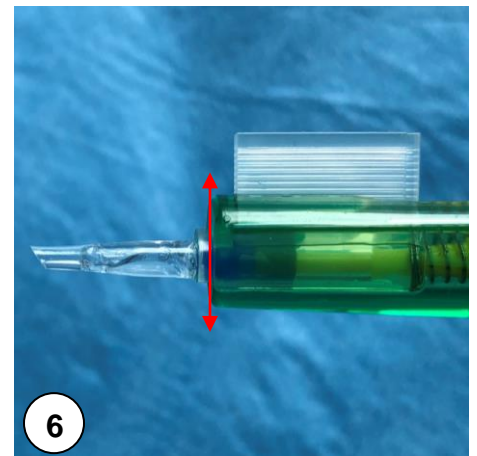

6

Place the cartridge into the injector. Push the plunger until the blue cushion reaches the tip, followed by short pressure relief. Implant the IOL.

